

# T.I.M.E. MultiTherapist

Versatile portable therapy unit combining

TENS, Interferential, MicroCurrent, and EMS.



## FEATURES

Small, hand held

Four treatment modes

Professional Power, but with sensitive intensity controls - 1mA steps (.01mA in MIC mode)

Mains or battery operation

TENS	Hz	uS	Special			
Constant	1-150 1Hz steps	50-300 10uS steps				
Burst	100	5-300	Burst rate 0.5-5Hz			
PW Modulation	1-150	50-300	Cycle time 5-30 s			
PR Modulation	1-150	50-300	Cycle time 5-30 s			
EMS	Hz	uS	Work	Rest	Ramp	Synch
	1-150	50-300	1-30	0-60	1-6	S/A/D
INTERFERENTIAL	Mode	Hz	Cycle time			
P1	Sweep	1-10	5-30 s			
P2	Sweep	1-150 Hz	5-30 s			
P3	Sweep	80-150 Hz	5-30 s			
P4	Sweep	Set to 160% of set	5-30 s			
MIC		Hz	mS	Cycle time		
P1	Constant	1-150 1Hz steps	2-200 ms in 1ms steps			
P2	Width Modulation	1-150	2-200 ms	5-30 s 60% decrease.		
P3	Rate Modulation	1-150	2-200 ms	5-30 s 60% decrease.		

The Safety Lock Feature automatically activates after there is no operation in the panel for 30 seconds by locking out the ability to press the buttons. This is a safety feature to prevent accidental changes to your settings and to prevent accidental increases to the intensity levels. You can press either one of the [▼] buttons to unlock the device.

TensCare Ltd  
9 Blenheim Road  
Epsom  
Surrey KT19 9BE  
01372 723434

